The logo features the word "JET" in a bold, white, sans-serif font, with a small "TM" trademark symbol to its upper right. Below "JET" is a smaller, lowercase "c" in a white, serif font. This text is centered within a teal circle. The background of the entire page is a pattern of overlapping circles in various shades of green and teal, arranged in a way that suggests a dynamic, upward-sloping path.

JETTM
c

The word "cache" is written in a white, lowercase, serif font, with a small "TM" trademark symbol to its upper right. It is centered within a solid teal horizontal bar that spans the width of the page.

cacheTM

READY TO USE JET? LET'S GET STARTED.

JET is a ClO₂ Sporidical foam for the disinfection of non-porous medical surfaces and non-invasive medical devices. Not intended to be used with ultrasound probes or invasive medical devices.

PRE-USE CHECKS

Some important reminders before starting:

Always read the label and product information before use. Disinfect hands and wear the appropriate personal protective equipment (PPE) when handling disinfectants and medical devices.

Do not use if product is damaged. JET is for professional use only. Do not apply to brass, copper, aluminium, mild steel, or silver surfaces.

Dispose of the empty container and used wipes in accordance to local health policies. Do not reuse the empty container. Do not reuse wipes.

To assemble JET, take a trigger, unscrew the caps on the bottles, and following the red A and blue B indicators on the bottle label, insert the tubes into the respective bottle necks.

Fasten the levers anti-clockwise on each side of your trigger to secure. Ensure the nozzle is aligned to the top line on the trigger head. JET is now ready to use.

HOW TO USE

Apply at least 2 doses onto a wipe, to cover an area of 50cm x 20cm (0.1m²)

Wipe onto the surface.

Ensure that the surface remains wet for the 1-minute contact time. Leave to air dry.

HIGH-PERFORMANCE CHEMISTRY

JET is based on a proprietary chlorine dioxide (ClO_2) chemistry, a well-documented and highly effective biocide. JET incorporates two separate compartments that contain Base solution (citric acid) and Activator Solution (sodium chlorite). When mixed by pulling the JET Trigger, chlorine dioxide (ClO_2) chemistry is generated. JET contains a non-ionic surfactant for added cleaning performance.

BIOCIDAL PERFORMANCE

JET is efficacious against bacterial spores, mycobacteria, viruses, fungi and vegetative bacteria in a contact time of 1 minute. The product has been tested according to the relevant European Standards required for disinfectants used in medical areas.

STORAGE CONDITIONS

JET can be kept for 2 years before being used, and should be stored in a cool dry place, out of direct sunlight, between 10°C – 35°C .

Base Solution

Warning

Causes serious eye irritation.

Precautionary statements

Wear eye protection.

IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

UFI: Base Solution: KE90-00M5-JOOD-UNMU

If you believe that this device has contributed to a serious incident (i.e. a death, the temporary or permanent serious deterioration of a person's state of health, or a serious public health threat), contact the device manufacturer and your national competent authority immediately.

Manufactured by: Tristel Solutions Ltd.

For supporting documents such as safety data sheets, microbiological test data and reports, contact Tristel, your local distributor or visit www.tristel.com.

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IMPORTERS

EUROPEAN UNION (EU) / EUROPEAN ECONOMIC AREA (EEA) / NORTHERN IRELAND / SWITZERLAND

**AUSTRIA, BULGARIA, CROATIA, CZECH REPUBLIC,
ESTONIA, GERMANY, GREECE, HUNGARY, LATVIA,
LITHUANIA, POLAND, SLOVAKIA, SLOVENI**

Tristel GmbH

Karl-Marx-Allee 90A, 10243 Berlin, Germany

T +49 (0)30 54844226 - **E** deutschland@tristel.com

W www.tristel.de

BELGIUM, FRANCE, LUXEMBOURG, NETHERLANDS

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NORTHERN IRELAND

BRENNAN & CO NI

Unit 6 & 7, Carrowreagh Business Park, Dundonald, Belfast,
Northern Ireland, BT16 1QQ

T +44 2890 410 136 - **E** info@brennanconi.com

W www.brennanconi.com

SPAIN

VESISMIN, S.L.

C/ Aribau 230-240, 6ª planta, L-0, 08006 Barcelona, Spain

T +34 934 09 53 01 - **E** vesismin@vesismin.com

W www.vesismin.com

SWITZERLAND

TRISTEL AG

Sandgrube 29, 9050 Appenzell, Switzerland

T +41 (0)71 5670658 - **E** schweiz@tristel.com - **W** www.tristel.com

A series of horizontal dotted lines spanning the width of the page, intended for writing.

cache™

Tristel Solutions Ltd, Unit 1b Lynx Business Park, Fordham Road, Snailwell, Cambridgeshire, CB8 7NY, UK
T +44 (0)1638 721500 - E mail@tristel.com - W www.tristel.com

Tristel NV, Smalldlaan 14B,
2660 Antwerp, Belgium

Tristel AG, Sandgrube 29,
9050 Appenzell, Switzerland

Where necessary, external translations have been sourced from our ISO 17100:2015 approved supplier.

Brought to you by Tristel.

CE
2797