



**TANK**

c

**DETERGENT**

cache<sup>™</sup>

## READY TO USE TANK? LET'S GET STARTED.

TANK is a storage and distribution system fuelled by capsules. TANK provides a complete surface disinfection protocol for your hospital.

TANK Detergent is intended for the cleaning of non-porous surfaces.

### PRE-USE CHECKS

Some important reminders before starting: Use detergents safely. Always read the product label, safety data sheets (SDS) and product information before use and wear the appropriate personal protective equipment (PPE).


TANK Detergent is for professional use only. Before preparing your solution, make sure your TANK is empty. It should be located on a flat surface, close to a water source in a central location. Remember to check the tap is closed before filling!




# TANK

C


## HOW TO USE

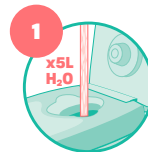


Fill TANK with ambient tap water until it reaches the 5L mark. Do not use hot water.

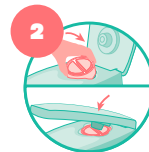

Take one TANK Detergent capsule and place it face down into the TANK lid. Close the lid to burst the capsule.

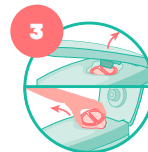

Open the lid and remove the now empty capsule. This can be rinsed out and sent for recycling.

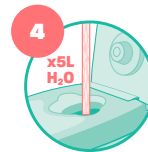

Fill TANK with additional ambient tap water until it reaches the 10L mark. Your TANK Detergent solution is now ready to use.



Grab a bottle, flip the tap to open, and fill with working solution. The reusable 500ml bottles can be paired with any of the available applicators.



Your working solution can be kept for up to 30 days within TANK or a 500ml bottle. To keep track of this, make a note of the time and date on the TANK lid and any bottles you decant the solution into.



Apply the working solution onto a wipe, or directly onto a surface, and wipe.

## REFILLING YOUR TANK

Before you refill your TANK, rinse it out with tap water.

## SMALL CAPSULE, BIG IMPACT

Single use plastic is rubbish. TANK's reusable plastic components are designed to last, and when they do reach the end of their life they can be recycled. But that's not all. By separating the solution from the wipe, TANK enables you to choose plastic-free wipes and use fewer of them, helping you reduce your plastic waste even further.

## STORAGE

TANK Detergent capsules can be kept for 2 years before being used, and should be stored in a cool dry place, out of direct sunlight, between 10°C - 35°C.

## HOW TANK DETERGENT WORKS

TANK Detergent capsules are filled with a non-enzymatic cleaning concentrate for dilution into 10 litres of water. TANK Detergent is ideal for cleaning and damp dusting. It leaves surfaces looking clean, shiny, and smear-free.

Many guidelines highlight cleaning as the most important part of a decontamination protocol. Without cleaning, soil and organic matter can create a barrier between a disinfectant and microorganisms, preventing disinfection.

## CONTENTS

- 200ml Detergent Solution

### **Danger**

Causes skin irritation.  
Causes serious eye damage.

### **Precautionary statements**

Wear protective gloves/protective clothing/eye protection/face protection.

**IF ON SKIN:** Wash with plenty of water.

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

If skin irritation occurs: Get medical advice/attention.



TANK Detergent capsules are marketed in compliance with the EU and GB Detergents Regulations.

**UFI:** EV90-J025-200V-4CJ5

**TEMPERATURE LIMIT****KEEP AWAY FROM SUNLIGHT****DO NOT USE IF PACKAGE IS DAMAGED****CONSULT INSTRUCTIONS FOR USE****MANUFACTURER****USE BY DATE****BATCH CODE****DO NOT RE-USE**

# cache<sup>TM</sup>

**Cache is a collection of products for the cleaning and sporicidal disinfection of surfaces brought to you by:**



Tristel Solutions Ltd, Lynx Business Park, Fordham Road, Snailwell, CB8 7NY, UK  
T +44 (0)1638 721500 - E mail@tristel.com - W www.tristel.com

**Australian Sponsor:** Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192, AU  
T 1 300 680 898 - E mail-au@tristel.com - W www.tristel.com

**New Zealand:** Tristel New Zealand Ltd, 23 Birch Avenue, Tauranga 3110, NZ  
T +64 (0)7 5771560 - E info@tristel.co.nz - W www.tristel.com

CCH/TANK/025- V2 - 11/MAY/2022

