

A CHLORINE DIOXIDE (CIO₂) SPORICIDAL FOAM FOR THE DISINFECTION OF NON-POROUS SURFACES



bottle label, insert the tubes into the respective bottle necks.





Apply at least 2 doses onto a wipe.



Wipe onto the surface.



Ensure the surface remains wet for 1 minute. Apply additional solution as needed. Leave to **dry**.



When you can't recycle, reuse! Read the 'BIG IMPACT' section

PRE-USE CHECKS

Some important reminders before starting:

- Use disinfectants (product type 2) safely.
- Always read the label and product information before use.
- Wear the appropriate personal protective equipment (PPE).
- JET is for professional use only.
- Do not apply to brass, copper, aluminium, mild steel, or silver surfaces.
- Ensure the surface is free from heavy soiling before use.



Refer to the User Guide for full instructions.

BIG IMPACT

Your JET trigger should be reused with the second JET bottle. To do this, unfasten the levers clockwise to remove the trigger. Before following the assembly instructions above, give the dip tubes a rinse with tap water and dry with a paper towel. After use, empty JET bottles and caps can be sent for recycling.

STORAGE CONDITIONS

JET can be kept for 2 years before being used, and should be stored in a cool dry place, out of direct sunlight, between 10°C - 35°C.

Once opened, JET has an in-use life of 3 months. You can track this by noting the day you opened JET on the bottle label.





Use biocides safely. Always read the label and product information before use